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1. Goal

To describe how EDTA plasma for exosome isolation must be prepared and stored for the purposes of the Radboud Biobank, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it.

1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Radboud Laboratory for Diagnostics.

2. Protocol EDTA plasma for exosome isolation

Collection

| | |
|--------------|----------------------------------------------------------------------------------------------------------|
| Delivery | In collection tube. |
| Type of tube | Standard 10 ml tube EDTA K2E, without gel, without protease inhibitors. BD tube 367525 or equivalent. |
| Temperature | Collect on ice and keep cool (4°C). |

Preparation

| | |
|---------------------|-----------------------------------------------------------------------------------------|
| Time until freezing | Prepare and freeze the collected blood as soon as possible. Maximum: within 4 hours. |
| Centrifuge | g-force: 2000g. Time: 20 minutes. Temperature: 4°C. |

Storage

| | |
|--------------|----------------------------------------------------------------------------------------------------------------------------------|
| Register | Register the microtubes in the Biobank management system and attach a cryovial label. |
| Aliquotation | Store the biomaterial in 0.5 ml aliquots to a maximum of 6 aliquots. If there is biomaterial left over add it to the 6 aliquots. |
| Type of tube | 2 ml polypropylene microtube and screw-top with O-ring. GREI722301UMC and GREI368380UMC or equivalent. |
| Temperature | -80°C |

Miscellaneous

| | |
|------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Deviations | Note all deviations from this procedure in the Biobank management system. Assess the samples macroscopically for the following deviations (Haemolytic, Icteric and Lipemic) and register them in the Biobank management system. |
|------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

3. Quality assurance

Review takes place via an internal audit (see the audit plan).

3.1 Performance indicator

95% of the relevant samples of the sub biobank have been prepared and stored within the parameters of this procedure.

4. Comments

-

5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

| link | Title |
|-----------------------------|-----------------------------------------------------------------------|
| Qdoc 045014 | Radboud Biobank voorschrift RLD (NB only accessible to RLD employees) |
| Qdoc 078088 | Monstermanagement RLD t.b.v. Radboud Biobank |

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