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07-01-2020 Title is changed

07-01-2020 Performance indicator is changed

26-03-2019 Time until preparation is changed in time until freezing

26-03-2019 Biobank tracking system is changed in Biobank management system

1. Goal

To describe how CSF must be prepared and stored at the Radboud Biobank for the purposes of the Parkinson op Maat project, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it.

1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Radboud Laboratory for Diagnostics.

2. Protocol CSF collected in CSF tube (polypropylene) - POM

Collection

Delivery In 1-3 CSF tube(s).

Type of tube CSF tube (polypropylene) 10 ml.

Volume Minimum of 3 ml, the goal is 20 ml (maximum).

Temperature Keep tubes at room temperature until prepared for storage.

Preparation

Time until freezing Maximum 2 hours.

Cell count Use 1 ml from the tube with the clearest CSF for the cell count. Centrifuge the rest including all tubes.

Centrifuge
g-force: 2000g.
Time: 10 minutes.
Temperature: Room temperature.

Preparation protocol Combine supernatant from all tubes in a 50 ml polypropylene tube and homogenize by inverting the tube 10 times.

Storage

Registration Register the microtubes in the biobank management system and attach a cryovial label.

Aliquoting Store the biomaterial in aliquots of minimal 0.5 ml (maximum 38 aliquots).

Type of tube 2 ml polypropylene microtube and screw-top with O-ring.
GREI722301UMC and GREI368380UMC or equivalent.

Temperature -80°C

Miscellaneous

Comment By traumatic epidural puncture the tube contaminated with blood will be thrown away at collection and will not be delivered.

Deviations Note all deviations from this procedure in the biobank management system.

3. Quality assurance

Review takes place via an internal audit (see the audit plan).

3.1 Performance indicator

95% of the samples of the sub biobank have been prepared and stored within the parameters of this procedure.

4. Comments

-

5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

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