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N.A.

1. Goal

To describe how heparin-whole blood must be prepared and stored for the purposes of the Radboud Biobank, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it

1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Cell Culture facility of Genome Research of the Radboudumc.

2. Protocol heparin-whole blood for EBV-cellines.

Collection

Delivery	In collection tube.
Type of tube	4 ml lithium-heparin tube. No gel, no protease inhibitors BD tube 367883 or equivalent.
Temperature	Keep the tubes at room temperature until preparation.

Preparation

Time until freezing	Within 36 hours after collection.
Protocol	Add 2 ml cold fetal calf serum (FCS) and 0,5 ml DMSO to 2,5 ml heparin blood, mix, immediately cool by placing on ice, divide into aliquots immediately.

Storage

Register	Register the microtubes in the Biobank management system and attach a cryovial label.
Aliquotation	Divide material into 3 aliquots (1,5 ml each). When there is less than 2,5 ml blood use the same volume of FCS/DMSO-mix and divide into 1 or 2 aliquots.
Type of tube	2 ml polypropylene micro tube and screw-top with O-ring GREI722301UMC and GREI368380UMC or equivalent.
Temperature	Freeze at -80°C for at least 24 hours and a maximum of 7 days (lower the temperature slowly and in a controlled manner). Samples are stored long term in liquid nitrogen.

Miscellaneous

Deviations	Note all deviations from this procedure in the Biobank management system
------------	--

3. Quality assurance

Review takes place via an internal audit (see the audit plan).

3.1 Performance indicator

95% of the relevant samples have been prepared and stored within the parameters of this procedure.

4. Comments

-

5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

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