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Revision changes

1. Goal

To describe how EDTA plasma for cell-free DNA must be prepared and stored for purposes of the Radboud Biobank, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it.

1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Radboud Laboratory for Diagnostics.

2. Protocol EDTA plasma for cell-free DNA

Collection

| | |
|--------------|-------------------------------------------------------------------------------------------------------|
| Delivery | In three collection tubes. |
| Type of tube | Standard 10 ml tube EDTA K2E, without gel, without protease inhibitors. BD tube 367525 or equivalent. |
| Temperature | Keep tubes at room temperature until prepared for storage. |

Preparation

| | |
|---------------------|----------------------------------------------------------------------------------------------------------------------------------|
| Time until freezing | Prepare and freeze the collected blood as soon as possible. Aim: within 2 hours after collection. Maximum: within 4 hours. |
| Centrifuge | g-force: 2000g. Time: 10 minutes. Temperature: Room Temperature. |
| Processing | Bring the supernatant of each tube over to a 5 ml polypropylene centrifugation tube (Eppendorf No. 0030119.401 or equivalent). |
| Centrifuge | 5 ml polypropylene centrifugation tubes. g-force: 16.000g. Time: 10 minutes. Temperature: Cooled 4 °C. |
| Processing | Aliquotation of Supernatant. |

Storage

| | |
|--------------|--------------------------------------------------------------------------------------------------------|
| Register | Register the 5ml tubes in the Biobank management system and attach a cryovial label. |
| Aliquotation | Bring the supernatant of each centrifugation tube over to a 5 ml polypropylene storage tube. |
| Type of tube | 5 ml polypropylene tube with external thread and screw cap. (Fisherbrand No. 10-500-27 or equivalent). |
| Temperature | -80°C. |

Miscellaneous

Deviations Note all deviations from this procedure in the Biobank management system.

3. Quality assurance

Review takes place via an internal audit (see the audit plan).

3.1 Performance indicator

95% of the relevant samples have been prepared and stored within the parameters of this procedure.

4. Comments

-

5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

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