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Revision changes

1. Goal

This SOP is intended for Biobanks that collect blood for cell-free DNA outside the RadboudUMC. To describe how EDTA plasma for cell-free DNA must be prepared and stored for purposes of the Radboud Biobank, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it.

1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Radboud Laboratory for Diagnostics.

2. Protocol EDTA plasma for cell-free DNA

Collection

Delivery	In three collection tubes. Do not send the tubes to the laboratory with the pneumatic tube system but use intern transport.
Type of tube	CE-IVD Cell-Free DNA collection tube, Roche Diagnostics 7832389001 or equivalent.
Temperature	Keep tubes at room temperature until prepared for storage.

Preparation

Time until freezing	Prepare and freeze the collected blood within 7 days of collection.
Centrifuge	g-force: 2000g. Time: 10 minutes. Temperature: Room Temperature.
Processing	Bring the supernatant of each tube over to a 5 ml polypropylene centrifugation tube (Eppendorf No. 0030119.401 or equivalent).
Centrifuge	5 ml polypropylene centrifugation tubes. g-force: 16.000g. Time: 10 minutes. Temperature: Cooled 4 °C.
Processing	Aliquotation of Supernatant.

Storage

Register	Register the 5ml tubes in the Biobank management system and attach a cryovial label.
Aliquotation	Bring the supernatant of each centrifugation tube over to a 5 ml polypropylene storage tube.
Type of tube	5 ml polypropylene tube with external thread and screw cap. (Fisherbrand No. 10-500-27 or equivalent).

Temperature -80°C.

Miscellaneous

Deviations Note all deviations from this procedure in the Biobank management system.

3. Quality assurance

Review takes place via an internal audit (see the audit plan).

3.1 Performance indicator

95% of the relevant samples have been prepared and stored within the parameters of this procedure.

4. Comments

-

5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

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