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06-01-2020: Performance indicator is changed

26-03-2019: til is changed in until

26-03-2019: biobank tracking system is changed in Biobank management system

1. Goal

To describe how feces for the Parkinson op Maat (POM) project must be prepared and stored within in the Radboud Biobank, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it.

1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Radboud Laboratory for Diagnostics.

2. Protocol feces POM

Collection

Delivery Feces container with spoon.

Type of container Feces container with spoon 69x34 mm.

Temperature Keep containers at 4°C until preparation.

Preparation

Time until freezing Maximum of 72 hours after collection provided it has been kept at 4°C.

Temperature 4°C (in refrigerator (4°C) or on ice).

Preparation protocol Homogenize feces with the using a wooden spatula. If the feces is too hard to homogenize do not homogenize but divide into portions and not the deviation in the biobank tracking system.

Storage

Register Register tubes with feces in the biobank management system and attach a cryovial label.

Aliquotation 12 aliquots of 1-2 grams should be stored. If there is not enough material store less aliquots. If the feces is too hard divide the material over the tubes (max. 12, volume may deviate).

Type of tube Polyethylene feces tube with a 19 mm diameter, top diameter is 23 mm

Temperature -80°C

Miscellaneous

Deviations Note all deviations from this procedure in the Biobank management system.

3. Quality assurance

Review takes place via an internal audit (see the audit plan).

3.1 Performance indicator

95% of the relevant samples from the sub Biobank have been prepared and stored within the parameters of this procedure.

4. Comments

-

5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

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