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Revision changes

06-01-2020 Performance indicator is changed
06-01-2020 Registration of deviations is changed
Comment Added MEN/Diabetes pearl cooled delivery/preparation.
21-03-2019 till changed in until
25-03-2018: Biobank tracking system is changed in Biobank management system

1. Goal

To describe how EDTA plasma (10 ml) must be prepared and stored within the framework of the Parelstoer Institute for the purposes of the Radboud Biobank, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it.

1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Radboud Laboratory for Diagnostics.

2. Protocol EDTA plasma 10 ml

Collection

Fasting patient	Dependant on the requirements of each Pearl/sub-biobank.
Delivery	In collection tube.
Type of tube	Standard 10 ml tube EDTA K2E, without gel, without protease inhibitors. BD tube 367525 or equivalent.
Temperature	Keep tubes at room temperature until prepared for storage.

Preparation

Time until freezing	Prepare and freeze the collected blood as soon as possible.. Aim: within 2 hours. Maximum: within 4 hours.
Centrifuge	g-force: 2000g Time: 10 minutes. Temperature: At room temperature.

Storage

Register	Register the microtubes in the Biobank management system and attach a cryovial label.
Aliquotation	Store the biomaterial in 0.5 ml aliquots to a maximum of 6 aliquots. If there is biomaterial left over add it to the 6 aliquots.
Type of tube	2 ml polypropylene microtube and screw-top with O-ring. GREI722301UMC and GREI368380UMC or equivalent.
Temperature	-80°C

Miscellaneous

Comments	The tube is delivered on ice and is prepared at 4°C for the DIABETES and MEN pearls.
----------	--

Deviations Note all deviations from this procedure in the Biobank management system.
Assess the samples macroscopically for the following deviations
(haemolytic, Icteric and Lipemic) and register them in the Biobank
management system.

3. Quality assurance

Review takes place via an internal audit (see the audit plan).

3.1 Performance indicator

95% of the relevant samples from the sub Biobank have been prepared and stored within the parameters of this procedure.

4. Comments

-

5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

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