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07-01-2020 Performance indicator is changed

07-01-2020 Deviations is changed

26-03-2019 biobank tracking system is changed in Biobank management system

1. Goals

To describe how serum in cryovials must be prepared and for the purposes of the Radboud Biobank, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it

1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Radboud Laboratory for Diagnostics.

2. Protocol serum in cryovial

Collection

Delivery Divided into aliquots in 2 ml microtube.

Preparation

Time until freezing Prepare and freeze the collected blood as soon as possible.
Goal: within 2 hours.
Maximum: within 4 hours.

Storage

Register Register the Microtubes in the Biobank management system and attach a cryovial label.

Type of tube 2 ml polypropylene microtube and screw-top with O-ring.
GREI722301UMC and GREI368380UMC or equivalent.

Temperature -80°C

Miscellaneous

Deviations Note all deviations from this procedure in the biobank management tracking system. Assess the samples macroscopically for the following deviations (Haemolytic, Icteric and Lipemic) and register them in the Biobank management system.

3. Quality assurance

Review takes place via an internal audit (see the audit plan).

3.1 Performance indicator

95% of the relevant samples of the sub biobank have been prepared and stored within the parameters of this procedure.

4. Comments

5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

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