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## Revision changes

07-01-2020 Performance indicator is changed  
07-01-2020 Registration of deviations is changed

### 1. Goal

To describe how EDTA plasma in cryovials must be prepared and stored for the purposes of the Radboud Biobank, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it.

#### 1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Radboud Laboratory for Diagnostics.

## 2. Protocol EDTA-plasma in cryovial

### Collection

Delivery In microtube 2 ml (after aliquotation).

### Preparation

Time until freezing Prepare and freeze the collected blood as soon as possible..  
Aim: within 2 hours.  
Maximum: within 4 hours.

### Storage

Registration Register the microtubes in the biobank management system and attach a cryovial label.

Type of tube 2 ml polypropylene microtube and screw-top with O-ring.  
GREI722301UMC and GREI368380UMC or equivalent.

Temperature -80°C

### Miscellaneous

Deviations Note all deviations from this procedure in the biobank management system. Assess the samples macroscopically for the following deviations (Haemolytic, Icteric and Lipemic) and register them in the Biobank management system.

## 3. Quality assurance

Review takes place via an internal audit (see the audit plan).

### 3.1 Performance indicator

95% of the relevant samples from the sub Biobank have been prepared and stored within the parameters of this procedure.

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